Knockdown Bath Bench with Arms

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General Warnings

To ensure the safe installation, adjustment, and operation of the Bath Bench, these instructions **MUST** be followed.

FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.

If you are unable to understand these warnings, cautions, and instructions, contact a healthcare professional or dealer before attempting to use this equipment.

Warnings

- ⚠ **DO NOT** assemble and use this product without first reading and understanding this user's guide, otherwise injury or damage may occur.
- ⚠ **DO NOT** use if components are missing or are damaged. Contact Medline at **1-800-MEDLINE (633-5463) | 1-800-268-2848**.
- ⚠ **DO NOT** exceed weight capacity of **350 lb (158 kg)**.
- ⚠ **DO NOT** make any adjustments while using the product.
- ⚠ **DO NOT** stand on the product.
- ⚠ **DO NOT** excessively twist or turn while seated on the chair as this may cause a loss of balance and/or damage to the product.
- ⚠ **DO NOT** attempt to use the arms to support full body weight.
- ⚠ **DO NOT** attempt to use the backrest to support full body weight.
- ⚠ **DO NOT** use product for anything other than its intended use.
- ⚠ **DO NOT** use product if wobbly, unstable, or not level.
- ⚠ **DO NOT** use product on uneven surfaces.

Claims

Users with limited physical strength and/or mobility should be supervised and/or assisted while using the product. Prior to use, ensure the following:

1. Ensure chair is completely unfolded before use. Failure to do so may result in possible injury or damage.
2. All push buttons are fully protruded and are properly engaged.
3. There are no loose, broken or missing components.
4. All parts are secure.
5. Ensure the chair remains balanced.
6. Excessive twisting or turning while seated on the chair may cause a loss of balance and/or damage to the product.
7. All four legs touch the floor simultaneously at all times.
8. User evenly distributes weight between the arm supports when transferring in and out of the chair.
9. The tips are not worn, broken or missing. Replace any missing or damaged tips before use.
10. Ensure all legs are at the same height adjustment level before use.

Parts Listing

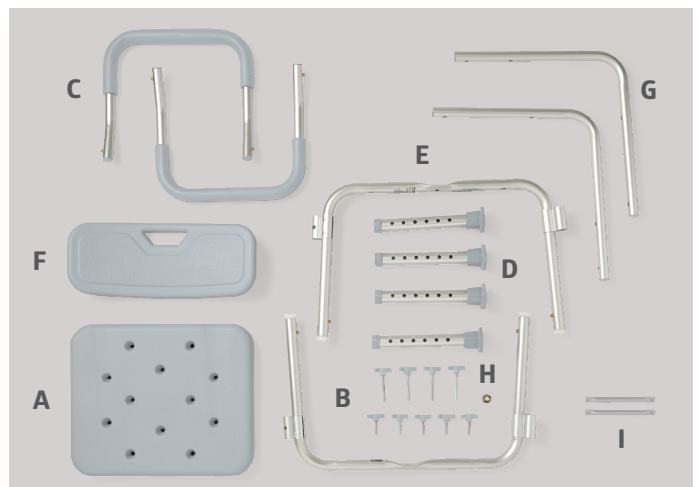
If exposed to cold temperatures, allow contents to come to room temperature before completing assembly. Remove contents from box, ensuring all parts listed are included:

G2-100KHX1:

- A.** Bench (imprinted with an A and B) (1)
- B.** Hand Screws for G2-100KHX1 (9)
- C.** Padded Arm Rests (2)
- D.** Aluminum Extension Legs (2)
- E.** Aluminum Main Frames (imprinted A and B) (2)
- F.** Backrest (1)
- G.** Aluminum Backrest Frames (2)
- H.** Curved Washer (1)

G2-200KHX1:


- A.** Bench (imprinted with an A and B) (1)
- B.** Hand Screws for G2-200KHX1 (5)
- C.** Padded Arm Rests (2)
- D.** Aluminum Extension Legs (2)
- E.** Aluminum Main Frames (imprinted A and B) (2)
- H.** Curved Washer (1)
- I.** Reinforcement Bars (2)

Assembly Instructions

Seat's Main Frame Assembly

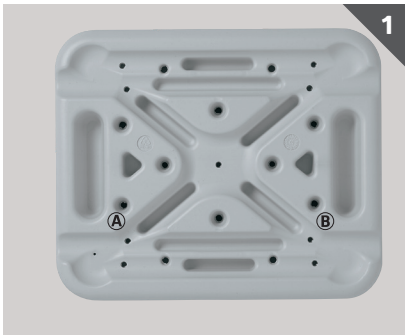1. Place the seat bench upside down on top of a raised flat surface to reveal the pre-drilled holes and indentations for setting in the seat's main frame.
2. You should have received a total of 9 hand screws, 5 short ones and 4 long ones (for G2-100KHx1). You should have received a total of 5 short hand screws (for G2-200KHx1).
3. Locate the imprinted "A" indentation on the underside of the bench. Insert the #A main frame into the preformed indentation labeled #A.
4. Locate the imprinted "B" indentation on the underside of the bench. Insert the #B main frame into the pre-formed #B indentation.
5. Secure the center cross section using curved washer and one of the five short hand screws.

6. Lay the bench on its side.
7. (For G2-200KHx1 only) Attach the reinforcement bars to the bottom of the bench. Use the 4 short hand screws to attach the reinforcement bars. Insert each hand screw through the reinforcement bar and leg tube.
8. (For G2-100KHx1 only) Attach the backrest tubes to the bottom of the bench. Use the 4 long hand screws to attach the tubes. Insert each hand screw through the backrest tube and leg tube.

9. Place a leg extension piece on each leg. Make sure that all 4 legs are adjusted to same height.
10. Stand the chair upright. Attach the backrest pad to the backrest tubes using the remaining 4 short screws.
11. Insert one arm rest on each side. The arm rest should lean to the outside of the chair.
12. Your chair should now be complete.

Attach the Aluminum Extension Legs to the Aluminum Main Frame

1. Depress the pushbuttons on the main frame and place the aluminum extension legs into the main frame tube until secured at the desired height. Repeat with the three remaining legs. Ensure all four extension legs are installed at the same height and the pushbuttons are fully protruded before using.

Note: Improperly inserting the legs of the backrest may cause damage to the product. When the legs have been installed to the seat bench, the legs of the backrest will hangover the side of the table and are pointing towards the floor.

Installing and Assembling the Backrest: (Only pertains to G2-100KHX1)

1. Attach the backrest tubes to the bottom of the bench. Use the 4 long hand screws to attach the tubes. Insert each hand screw through the backrest tube and leg tube. (Figure 8-a & 8-b)
2. Place the bench right side up to assemble the backrest to the aluminum backrest frame.
3. Grab the backrest by the handle and position it into place fitting the pre-formed indentations located on the backside of the backrest so that the screw holes line up to the predrilled holes of the aluminum backrest frame. Attach and secure the hand screw. Once all four screws are in place, tighten for safety.

Assembling the Arm Rests

1. Insert each padded arm rest on either side of the seat bench. Ensuring the curved bar extends outward from the seat bench. The pushbuttons should lock securely into place.

Leg Height Adjustment

1. Before use, adjust the height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes.
2. Ensure that the legs are locked into place by pushing down and pulling up on each leg.

Cleaning

1. Clean with a non-abrasive cleaner or household cleaner with warm water.
2. Rinse with warm water and wipe dry after cleaning.
3. **DO NOT** use abrasive powders, cleaners, cloths, or scouring pads.

Maintenance

Periodic visual inspection of this product is recommended to ensure that all parts and hardware are secure, that components are in good working order and not cracked, broken or fractured, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at **1-800-MEDLINE (633-5463) | 1-800-268-2848**.

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Limited Warranty

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as tip accessories, casters and grips, which are subject to normal wear and replacement.

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Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2023 Medline Industries, LP. Medline is a registered trademark of Medline Industries, LP.

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